# Rubis

Hygiene, comfort and safety for bed bound individuals

# Mobile Shower System

### REF. 3017.00

The Rubis Shower in Bed System is personal hygiene at its most portable. Designed for individuals who are confined to bed, the fully contained mobile showering unit and easy drain waterproof bed sheet, allow clean, efficient, comfortable showering - no matter how immobile the individual.





Rubis 3017.00 + "Patient toilet" option réf 3017.60





### Rubis

Step 1



Firstly, fill Rubis's water container.

Step 2



Position Rubis at the end of the bed and select temperature. Press the heater button and wait for Rubis to reach desired temperature (this takes approximately ten minutes from cold)



Step 3

Lay one half of the purpose designed, double-width towel over the mattress, and allow the other half to rest over the side rails.

### Step 4



Next, fit the waterproof sheet under the patient. Position the sheet as evenly as possible using the fasteners provided, and fix to side rails and bed ends to ensure water containment.

Step 5



Unclip the shower hose in readiness to use. Then unclip the water vacuum hose and position it at the lowest point of the mattress where water pools fastest. Remove the patient's clothing and check the waters output temperature prior to application.

# Step 6



When the water temperature has reached the desired preset temperature it can then be applied to the patient.



With the shower procedure complete the patient can be dried and the sheet removed and stored. The patient can then be dressed.



### Step 8



ne procedure complete Rubis can then be emptied of waste water, and cleaned in readiness for the next use.



Reval® Continuing Care Limited Unit 17 Brynmenyn Busines Centre St Theodore's Way - Brynmenyn CF32 9TZ BRIDGEND - WALES Tel: 01656 721 001 Fax: 01656 720 006 www.revalcc.co.uk



### **Rubis**











# **UNIQUE FEATURES**

- Generous water flow and powerful drainage pump.

- Easy clean unit coated in ABS PMMA.

- Automatic washing and disinfection of the circuit and waste water tank.
- Thermic disinfection procedure.

# **Mobile Shower System**

### **REF. 3017.00**



### DIMENSIONS:

 Overall: Height 106 cm / Width 94 cm / Depth 58 cm • Weight: 38.3kg

- 1 Control panel
- 2 Waste water tank
- 3 Fresh water tank
- 5 Washbasin (option 3017.60)
- 6 Stainless steel bowl (option 3017.60)
- 7 Stainless steel handrail
- 8 Maintenance access panel
- 9 Disinfectant tank access panel
- 10 Wheel with brake
- 11 Flow intensity adjustment valve
- 12 Cleaning shower
- 4 Single handle bar faucet (option 3017.60) 13 Fresh water tank drainage
  - 14 Waste water tank drainage
  - 15 Mattress water drainage hose
  - 16 230V electric feeding cable
  - 17 Filter on suction piping

The Rubis Shower in Bed System comprises: a mobile shower trolley with six directional wheels, flexible showering hoses, fresh water tank, integral waste water tank and disinfectant tank, waterproof bed sheet (width 120cm), five single use shower sheets for bed (width 90cm), four fasteners, as well as a bath towel (158x220cm).

Reval® Continuing Care Limited Unit 17 Brynmenyn Busines Centre St Theodore's Way - Brynmenyn CF32 9TZ BRIDGEND - WALES Tel: 01656 721 001 Fax: 01656 720 006 www.revalcc.co.uk



#### DESCRIPTION:

- Steel frame, white colour, anti-corrosion coated.
- Covers in ABS PMMA.
- x6 Ø100mm directional wheels, 2 with brakes.
- 2 stainless steel handrails.
- Fresh water tank in stainless steel, capacity 23L.
- Waste water tank in PEHD, capacity 28L with wash nozzle integrated rotary.
- Disinfectant tank, capacity 5L.
- 24V shower pump, 8 litres/min
- 24V drainage pump, 14 litres/min Simultaneous pump function
- Temperature controller with double display
- (real temperature & set temperature)
- Red warning light (water too hot).
- · Level indicator for disinfectant tank.
- 230V 2700W Heating element
- Water sensor level for heating element.
- Integrated differential circuit breaker 16A 30mA.
- Electric components inside IPX5 waterproof box.
- IP68 waterproof light buttons.
- Handheld shower with flow adjustable trigger.
- Concealed disinfectant tank.
- Silicone autoclave piping.
- Sound: 50dBA / 46dBm

#### **DESCRIPTION OF SHOWER SHEET:**

- 13% PES 87% PVC
- 400gr/m<sup>2</sup>
- Fire standard : ISO14116: 1



### CONFORMITY:

- 93/42/CEE Medical Devbice Classe 1
- IPX4 Class 1 CEI 60601-1
- CE marking

(E

#### SUPPLIED ACCESSORIES:

- 3017.11 1 x Shower sheet for bed 120cm
- 3017.12 5 x Single use shower sheet for bed 90cm
- 3017.20 White Bath towel 158x220cm 100% coton 400gr/m<sup>2</sup>
- Four fasteners



### **Rubis**



Material: polyurethane & woven fabric.



Single use shower sheet for bed.



**ADDITIONAL ACCESSORIES** 



Rubis tap filling adapter (1 adapter, 1 push connector & 1 x 1m50 flexible)



Set of 2 dispensers: 1 for the shampoo & 1 for the washing product.







- "Patient's toilet" Option including: Washbasin with integrated drainage & siphon. Single handle bar faucet.

Reval® Continuing Care Limited Unit 17 Brynmenyn Busines Centre St Theodore's Way - Brynmenyn CF32 9TZ BRIDGEND - WALES Tel: 01656 721 001 Fax: 01656 720 006 www.revalcc.co.uk











Single use washing bowl in cविमोंगअभ्रुकी basin in ABS Packaging: 100 units for shampooing and











Foot wash basin in ABS for foot care and hygie-ne of ankles and heels.